

Black Ice

be cool. be very cool.®

The Black Ice NTX CoolTherapy System for the neck Designed to help relieve pain and reduce swelling.

Use this information to get the most out of Black Ice.

Recharge time

Ice Water: 1.5hr.

Freezer: 1.5hr.

Refrigerator: 4hr.

*Pack is fully charged when "solid."
For best results, keep packs flat
as possible while charging.*

The CTX CoolTherapy Pack easily attaches to any CoolSwap wrap with hook and loop fasteners.



INSB3

How long will it last?

A single fully charged Black Ice CTX CoolTherapy Pack lasts up to 1.5 hours—depending on user activity level and ambient conditions.

Uninterrupted relief? Absolutely.

Looking for long term relief from pain and swelling? The NTX comes with two CTX CoolTherapy Packs, allowing you to wear one while the other is recharging. With a 52°F output, you never have to worry about ice burn, so you can swap packs for as long as you need.

Storage

If you are not going to charge your CTX Personal Cooling Packs, store them flat and separately. **Get relief at a moment's notice.** Just store your cooling packs in the refrigerator or freezer year round.

Care

Hand wash packs and collar in dishwashing liquid and water. Rinse thoroughly, air dry.

Fits neck sizes 13¹/₂" - 18¹/₂"

Black Ice LLC

1255 Lynnfield Rd. • Suite 107 • Memphis, TN 38129 • 901.937.8129 • blackicecooling.com